## SoftPro® Static Knee Orthosis

### TREATMENT RATIONALE:

Facilitate maintenance or rehabilitation of the knee in treatment of joint stiffness or non-fixed adaptive tissue shortening contractures of the knee joint. A static brace that may be used to maintain end range or to provide incrementally restore range of motion through progressive extension stretching with frequent brace modification.

### FUNCTIONAL OBJECTIVES:

Maintain or increase elbow Range of Motion (ROM), allowing for increased functional use of the affected joint. Functional use of the knee can be significantly improved by increasing ROM, allowing for greater independence in ADL'S. If function is not restored or significantly limited, the device should be used to maintain ROM as needed.

### ORTHOTIC TREATMENT AND FITTING:

1. Use PROM to passively stretch the affected joint capsule(s), connective tissue, tendons, and muscles.
2. Slowly and gently stretch the elbow to the point of noticeable resistance only (no discomfort). Hold for a minute to allow the extension release of the affected joint.
3. Set the knee orthotic hinges to provide the desired orthotic therapy. For a static setting to maintain comfortable end range, set the goniometer hinges at end range. For a progressive extension orthotic therapy setting, add approximately 10° of extension to the comfortable end range. When the device is applied, a gentle stretch will be applied to the affected joint.
4. Open the upper and bottom cuff flaps. The device label is always placed on the top cuff for ease in device orientation.
5. Holding the knee at its furthest point of extension, place the orthotic on the leg with the hinge at the midline of the knee joint.
6. Secure the top cuff flap over the upper thigh.
7. Secure the bottom cuff flap over the shin.
8. Place the kneecap directly over the middle of the patella (centered). Bring each of the four straps on the kneecap thru the corresponding "D" ring on the orthosis and loosely secure the kneecap
9. Incrementally tighten all four kneecap straps toward the center of the kneecap to slowly bring the knee into a stretched (extended) position. *When properly fitted, the kneecap straps will form an "X" over the center of the kneecap.* The "X" orientation of the kneecap straps is important to equally distribute the kneecap pressure across the entire knee cap.
10. Palpate the medial hamstring in the crease of the knee joint. It may be taut upon initial brace application. Re-palpate after several minutes. There should be a noticeable softening of the tissue after several minutes of wear.
11. Check all cuff flaps and the kneecap for pressure. Two fingers should be able to be inserted between the flap and the skin. Loosen flaps if necessary.
12. Determine wearing schedule per therapy evaluation and physician's order.
13. Incrementally increase wearing time per patient tolerance and patient care plan up to a maximum of six hours on per shift. A minimum of three to four hours of wear daily after the adaptation period is recommended for best results.
14. Release and check for skin redness or pressure or patient discomfort every two to three hours at a minimum. Remove the orthotic device immediately if significant redness or pressure is evident. Notify the appropriate staff member(s) immediately and document any significant redness or signs of adverse pressure or shear. Discontinue device use until the skin integrity issues are resolved, and the device is modified or the wearing schedule is altered to eliminate potential skin integrity problems.
15. Follow manufacturer's instructions for care of the orthotic device. Always inspect the device between applications to ensure the soft goods are properly in place, the device settings have not been altered, and the device has not been soiled or would not provide any other risk to the patient prior to application.
16. Check device settings for continued application of the desired amount of extension at least once a month. Re-adjust the hinges to maintain 5° to 10° of extension beyond the point of resistance to stretch as needed for progressive extension orthotic therapy.


### Laundry Instructions:

- Remove uprights.
- Fasten all hook and loop attachments on soft cover.
- Hand or machine wash, gentle cycle with mild detergent. DO NOT USE COMMERCIAL WASHERS OR HOT WATER.
- No bleach or fabric softener.
- Air dry.

### Hinges:

- Clean with mild detergent or alcohol
- Rinse with clean water
- Dry with soft cloth

**WARNING: The product should be fit by trained personnel. The product is designed for single patient use only in order to avoid cross contamination. Any substitution or removal of the product's parts voids the manufacturer's warranty. OCSI/NeuroFlex, Inc. will assume no liability if the above instructions are not followed.**

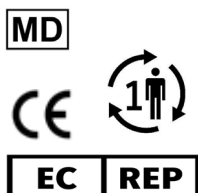Authorized Representative  
MDSS GmbH, Schiffgraben 41  
30175 Hanover, Germany

 Ongoing Care Solutions, Inc  
6551 43<sup>rd</sup> Street N., Unit 1403  
Pinellas Park, FL 33781  
800-375-0207 (P) 727-525-1424(F)  
[www.ongoingcare.com](http://www.ongoingcare.com)

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