REHOSOFT

This soft raised toilet seat facilitates the positioning of persons with difficulties to sit (hip surgery patients, elderly people, amputees).

Optimal Comfort is ensured by the technical properties of the material used:

- softness of the foam
- pleasant contact with the cover
- waterproof.

Installation and notice of use:

THE REHOSOFT WITH COVER MUST NOT BE HOLD BY THE COVER WHILE TAKING IT OUT ITS BOX

Simply put the REHOSOFT directly on the toilet bowls, REHOSOFT will be attached by compression. This product is designed for use with the most common types of toilet bowls. However it will not fit all types, so be sure that the unit is correctly installed, fits properly and is stable before use.

Always check for secure fit before using product. User's weight should be centered over toilet seat, not too forward or too far to one side, as unit may tip.

Specifications:

Dimensions: 420/400/170 mm (16.5'/15.7'/6.7')
Raising height: 115mm (4.5')
Weight: 2400 g (5.5 Lbs)
Materials: Polyurethane foam and sanitized PVC.
Maximum bearing weight: 120kg (265 Lbs)

Cleaning Instructions:

Soapy water and all usual products

Disinfection: Chlorine dilution; disinfectant aerosol

DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

BE CAREFUL NOT TO PUT THE BACK OF THE REHOSOFT UNDER WATER

LIMITED WARRANTY: (North America Only) Pharmaouest guarantees the REHOSOFT is free from defects in workmanship & materials for 1 year from the original purchase date. This applies when items are used for the purpose intended. In the event of a defect, please contact Customer service. (see phone number below). This warranty gives you right that may vary from State to State

WHAT IS NOT COVERED Wear & tear, aging, foam/item discoloring, flattening, density, variation, alteration, mishandling, faulty adjustment, misuse, improper care, accidents, rental use, obsolete items, service by anyone other than Pharmaouet, use of any non-Pharmaouest authorized parts, shipping damage, neglect, items sold 'as is' or damage due to natural acts are NOT covered

Manufactured by :

PHARMAOUEST INDUSTRIES, ACTIPOLE 2, BP 12, 35540 MINAC-MORVAN, FRANCE

Distributor:

Canada: AB4CD Inc. www.ab4cd.com +1 800 930 64 60

USA: MFM-Healthcare Inc. www.mfm-healthcare.com +1 888 883 40 64

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