SPORT Rehabilitator® Fitting and Application Instructions

Initial Fit

While seated and with the leg extended, measure down 4" below the center of the kneecap and make a mark at the center of the shin.

Applying the Brace

1. Begin by opening each strap on the lateral side of the brace. *(It is not necessary to open the patella strap)* To open, press in on the center of the buckle until the clip is released.
2. With the leg extended, place the brace on the leg with the upper edge of the shin cuff placed at the 4" mark.

AIR CONDYLE PLACEMENT IS IMPORTANT TO CUSTOMIZE EVERY FIT TO ENSURE OPTIMAL BRACE COMFORT AND PERFORMANCE.

Both air condyles need to be aligned precisely at the middle of the leg. If the air condyles do not align with the middle of the leg:

- a) Remove the hinge covers.
- b) Carefully detach the condyle pocket (secured to the upright with hook and loop).
- c) Re-attach the condyle pocket to be aligned precisely at the middle of the leg.
- d) Replace the hinge covers.

HINT – Each condyle pocket includes a contoured pad designed to accommodate the bony prominence of the knee. Aligned properly, optimal comfort will be achieved. The axis of the brace hinge should be aligned with the axis of the knee joint

3. Starting with the most distal bottom calf strap, secure the strap by inserting the tab at the end of the strap into the buckle on the cuff.
4. Next secure the most proximal upper thigh strap then the other bottom calf strap.
5. Stand and make sure the brace is fit properly with the shin cuff and air condyle pockets in the proper position.
6. Tighten the straps to comfort (in the same order as they were secured) to ensure that the brace doesn't slip. **Tighten patella strap if necessary**
7. Insert the clear end of the air actuator into the air receptacle on each side of the knee. Inflate each condyle until it applies a slight corrective force to the knee joint. Inflate to comfort.
8. Stand and take a short walk. Adjust the straps and inflation of the of the air condyles as necessary to ensure a comfortable and proper fit.

Removing the Brace

1. Before removing the brace, **ALWAYS REMOVE THE AIR** from the air condyles by inserting the black end of the air actuator into the air receptacle.
2. Starting with the bottom calf strap, undo the straps by releasing the buckles. Press in on the center of the buckle until the clip is released. Repeat this process for each strap in the same order as the straps were secured.

Flexion/Extension Control

If flexion or extension control is prescribed by physician, see enclosed packet with hinge setting tool (instructions included in packet).

Fitting Tips

1. Air Condyle Pockets are secured to the hinges with hook and loop. If any discomfort is felt on the bony prominence of the knee, remove hinge covers and simply loosen the condyle pad and pocket from the side bar and reposition to achieve a comfortable fit.
2. The brace comes with the foam contoured condyle pads inside the air condyle pockets. If the brace feels tight at the knee after fitting, the foam pads can be slightly reformed and/or completely removed to provide optimal comfort. To reform, remove from pocket and manipulate with hands to reform as necessary.
3. **EXTENSION ASSIST** - For your convenience, extension assist elastic bands (product accessory) have been applied to the brace. These bands may be quickly and easily removed for patients NOT needing an extension assist.

Contraindications

The Sport Rehabilitator® should not be applied if any part of the device comes in contact with an open wound. The patient must be ambulatory to benefit from the use of the Sport Rehabilitator®. The brace should not be applied if significant knee swelling negatively affects a properly fit brace.

Warning

The Sport Rehabilitator® should be fit by trained personnel to ensure that the device is fitted correctly. Wearing time should be determined by a clinician.

Cleaning the Brace

1. Thigh and shin cuff pads can be wiped down with a damp cloth and / or antibacterial wipe after wearing. If excessively soiled, remove pads from cuffs secured by hook and loop) and hand wash with mild detergent. Towel dry and lay flat until completely dry. **DO NOT PUT IN WASHER AND DRYER.**
2. Hinge covers can be hand washed in cool water with mild detergent and hung to dry. **DO NOT PUT IN WSHER AND DRYER.**
3. All straps can be removed from brace and hand washed in cool water with mild detergent and hung to dry. **DO NOT PUT IN WASHER AND DRYER.**
4. Gel Pad on patella strap can be hand washed in cold water with mild detergent. Towel dry and lay flat until completely dry. **DO NOT PUT IN WASHER AND DRYER.** Reapply to strap when completely dry.

Authorized Representative
MDSS GmbH, Schiffgraben 41
30175 Hanover, Germany

GUARDIAN BRACE, Inc
6551 43rd Street N., Unit 1403
Pinellas Park, FL 33781
727-526-0707 (P) 727-525-1424(F)
www.GuardianBrace.com

SPORT Rehabilitator® Fitting and Application Instructions

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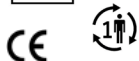
Limpiando Aparato Ortopédico

1. Thigh and shin cuff pads can be wiped down with a damp cloth and / or antibacterial wipe after wearing. If excessively soiled, remove pads from cuffs secured by hook and loop) and hand wash with mild detergent. Towel dry and lay flat until completely dry. DO NOT PUT IN WASHER AND DRYER.
2. Hinge covers can be hand washed in cool water with mild detergent and hung to dry. DO NOT PUT IN WASHER AND DRYER.
3. All straps can be removed from brace and hand washed in cool water with mild detergent and hung to dry. DO NOT PUT IN WASHER AND DRYER.
4. Gel Pad on patella strap can be hand washed in cold water with mild detergent. Towel dry and lay flat until completely dry. DO NOT PUT IN WASHER AND DRYER. Reapply to strap when completely dry.

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