medela 

# symphony® & lactina® breastpump kit

instructions

- ✓ for use with symphony & lactina breastpumps
- ✓ for double or single pumping

Not made with natural rubber latex. DEHP free

# Important Safeguards

See the instructions provided with your breastpump for additional safety precautions.

## **Warning: To avoid health risk and reduce the risk of injury**

- Breastpump kits are a personal care item intended for a single-user.
- Inspect the breastpump kit components prior to each use.
  - Do not use the components if they are damaged.
  - Do not use the components if dirt, mold, or other contamination is visible.
- Breastpump kit components that come in contact with breastmilk must be cleaned after each use.
- Do not modify any breastpump kit components.

## **Caution: Can lead to minor injury**

- Do not store wet or damp parts as mold may develop
- If tubing becomes moldy, discontinue use and replace tubing.
- Do not use the breastpump kit with any non-compatible breastpumps.
- If pumping is uncomfortable or causing pain, discontinue use and seek advice from a healthcare professional or breastfeeding specialist.
- Do not use the breastpump kit for any other purpose than expressing breastmilk.
- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- Take appropriate care in handling bottles and components.
- Do not use the breastmilk if bottles or components become damaged.

**SAVE THESE INSTRUCTIONS**

## Table of Contents

<b>1. Assembly &amp; Use.....</b>	<b>4-11</b>
1.1 Correctly fitting breastshields .....	4
1.2 Symphony® Pump Kit Assembly & Use .....	5-7
1.3 Lactina® Pump Kit Assembly & Use .....	8-9
1.4 Manual Breastpumping .....	10
1.5 How do I disassemble my kit?.....	11
<b>2. Cleaning .....</b>	<b>12-15</b>
2.1 After Each Use.....	12
2.2 Drying & Storing.....	13
2.3 Sanitize Daily.....	13
2.4 Piston/Cylinder Cleaning.....	14
2.5 Tubing Cleaning.....	14
<b>3. Storing breastmilk .....</b>	<b>15</b>
<b>4. Troubleshooting.....</b>	<b>16</b>
<b>5. Supplemental Information .....</b>	<b>17</b>

# 1 Assembly & Use

## Caution

While some discomfort may be felt when first using a breastpump, using a breastpump should not cause pain. If you are unsure about breastshield sizing, please contact a healthcare professional or breastfeeding specialist who can help you get a proper fit.

### 1.1 Correctly fitting breastshields

Properly fitting breastshields will ensure comfort and efficient milk expression. To determine if you have the correct size, look at the following diagram:

- The nipple should be centered in the tunnel of the shield.
- The nipple should move freely without rubbing when the pump is turned on.
- Minimal or no part of your areola should be pulled into the tunnel of the breastshield.
- You should see a gentle, rhythmic motion in the breast with each cycle of the pump.
- After pumping, your breast should feel much less full with no areas of hardness.

Correct fit

Medela offers a range of breastshields from small to XX-large which are available at many local retailers or visit [www.shopmedela.com](http://www.shopmedela.com).

### PersonalFit™ Breastshields

**21 mm (Small)** Item 87072

**24 mm (Medium)** Item 87073

**27 mm (Large)** Item 87274

**30 mm (X-Large)** Item 87075

**36 mm (XX-Large)** Item 87084

#### Choosing the right size PersonalFit™ breastshield:

- 1 Determine the size you're currently using. If you're not sure, look for the size embossed on your breastshield (see picture). 24 mm (M) breastshields are provided with Medela breastpumps.

- 2 While pumping, compare your fit to the images below as a sizing guideline.

Correct fit

Your breastshield is **too small**; try a larger size


Your breastshield is **too large**; try a smaller size

## Caution


- Wash hands with soap and water prior to handling your kit.
- Always inspect breastshields, connectors, valves, membranes and tubing prior to use for cleanliness.
- Contact your healthcare professional or breastfeeding specialist if you can express only minimal or no milk or if expression is painful. See supplemental information section 5 for more information.
- Do not try and express with vacuum that is too high and uncomfortable (painful). The pain, along with potential breast and nipple trauma may decrease milk output. See section 5 for more information.

### 1.2 Symphony® Pump Kit Assembly & Use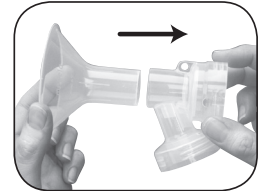


- 1 Securely attach breastshields to connectors.

## 1. Assembly & Use (cont.)

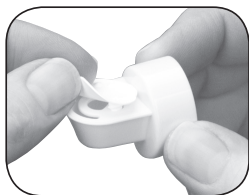
2 Snap white membranes onto yellow valves until membranes lie flat.

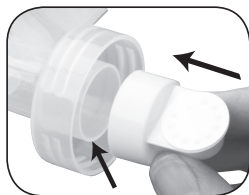
3 Push yellow valves onto inner rings of connector bottoms.


8 To open lid, hold down the oval button on the top of the Symphony breastpump and lift up on the handle.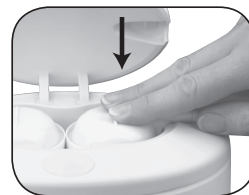

9 Push membrane caps into place on the pump. A "click" will be heard once properly seated. (**Note:** if single pumping, only seat one membrane cap.)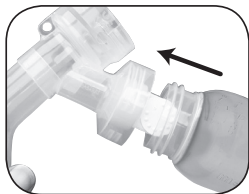

4 Screw collection containers onto connector bottoms.

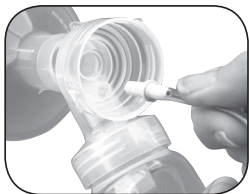
5 Insert pale yellow round ends of tubing into openings in back of connectors.

10 Close lid before pumping. (*If lid does not close, check to see that membrane caps are properly seated.*)

11 Center breastshields over your nipples and start pumping. (**Caution:** pushing shields too firmly onto breasts may affect milk flow.)

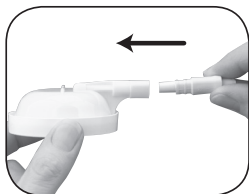6 Insert the triangular two-toned yellow ends of tubing into ports on membrane caps.

7 Fit protective membranes onto membrane caps.

12 To turn the pump on, press the power button. You will be starting in the Stimulation Phase. After 2 minutes, the pump will automatically change to the Expression Phase.

13 If let-down occurs before the Expression Phase begins, press the let-down button to begin the Expression Phase.

14 When your pumping session is over, store your breastmilk and clean your kit. See section 2 for cleaning instructions.

# 1. Assembly & Use (cont.)


## 1.3 Lactina® Pump Kit Assembly & Use

### ⚠ Caution


- Wash hands with soap and water prior to handling your kit.
- Always inspect breastshields, connectors, valves, membranes and tubing prior to use for cleanliness.
- Contact your healthcare professional or breastfeeding specialist if you can express only minimal or no milk or if expression is painful. See Supplemental Information section 5 for more information.
- Do not try and express with vacuum that is too high and uncomfortable (painful). The pain, along with potential breast and nipple trauma may decrease milk output. See section 5 for more information.


**Wash hands with soap and water prior to handling your kit.**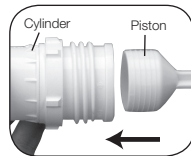


- 1** Slide the vacuum regulator ring onto cylinder. Push handle end of piston into cylinder.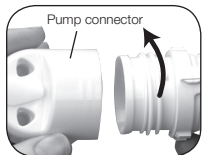


- 2** Screw cylinder tightly into Lactina pump connector.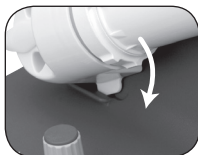


- 3** Attach pump connector to Lactina, and rotate cylinder down into position.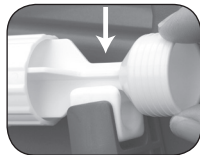


- 4** Slip piston into pumping arm cradle, and push down until snug.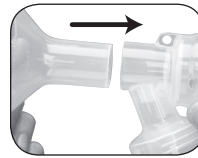


- 5** Securely attach breastshields to connectors.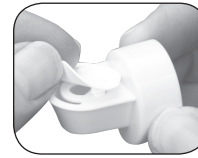


- 6** Snap membranes onto yellow valves until membranes lie flat.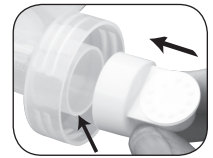


- 7** Push yellow valves onto inner rings of connector bottoms.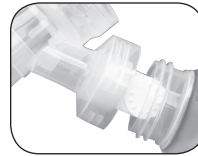


- 8** Screw collection containers onto connector bottoms.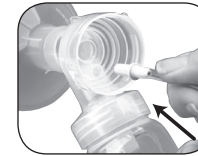

- 9** Insert pale yellow round ends of tubing into openings in back of connectors.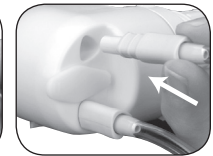


- 10** Insert the triangular two-toned yellow ends of tubing into ports on pump connector.

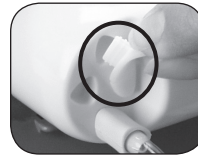- (Note: if single pumping, insert port plug into unused port.)**

- 11** Center breastshields over your nipples and start pumping. **(Caution: pushing shields too firmly onto breasts may actually affect milk flow.)**
- 12** Turn pump on and set vacuum regulator ring to "MIN." Rotate vacuum regulator towards max to adjust for comfort once pumping begins.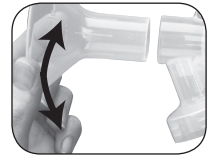


- 13** When your pumping session is over, store your breastmilk and clean your kit. See section 2 for cleaning instructions.

# 1. Assembly & Use (cont.)

## 1.4 Manual Breastpumping

Start by following instructions 1-8 from Lactina Pump Kit Assembly, then proceed with step 10 below.

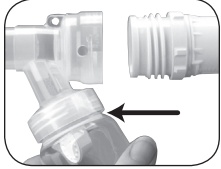**10** Screw cylinder into back of breastshield.

**11** Center breastshield over your nipple and start pumping by pushing and pulling on end of piston.

**12** When your pumping session is over, store your breastmilk and go to the section 2 for cleaning instructions.

## 1.5 How do I disassemble my kit?

**1** Finish pumping.

While the pump is still running, disconnect tubing from breastshields.


*(Let pump run for 2 minutes with tubing attached. This will eliminate condensation that naturally results from humidity.)*


**2** Remove collection container and store breastmilk.

**3** Disassemble.


All parts that come in contact with your breast and breastmilk must be disassembled and washed for the kit and pump to function efficiently.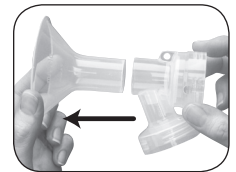

**a** Separate breastshields.

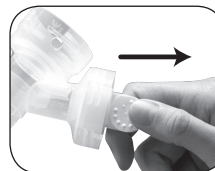**b** Twist and pull the yellow valves off the breastshields.

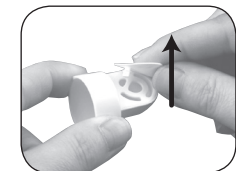**c** Remove the white membranes from the valves.

## 2. Cleaning


### Caution

- Only use drinking-quality water for cleaning.
- Separate and wash all parts that come in contact with the breast and breastmilk immediately after use to avoid accumulation of breastmilk residue and to prevent growth of bacteria.
- Do not store wet or damp parts as mold may develop.
- If mold develops in tubing, discontinue use and replace tubing. Contact Medela Customer Service at [customer.service@medela.com](mailto:customer.service@medela.com) or 1-800-435-8316.

### 2.1 After Each Use

- 1 Before cleaning, disassemble kit by following instructions in section 1.5.

Rinse all separated parts that came in contact with breast and breastmilk in cool water in order to remove breastmilk residue.

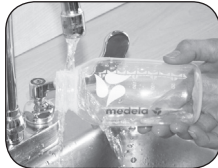- 2 **Washing in sink:**                      or                      **Washing in dishwasher:**
- Soak all separated parts in warm soapy water for 5 minutes.
  - Clean with a clean dishcloth or soft brush.
  - Rinse all separated parts with clear water.
  - Allow all parts to air dry in a clean area.
  - Store dry parts when not in use.
- Wash all separated parts on top rack of dishwasher.
  - Allow all breastpump parts to air dry in a clean area.
  - Store dry parts when not in use.

### 2.2 Drying & Storing

Allow all breastpump kit parts to air dry in a clean area and store dry parts when not in use. Do **NOT** store wet or damp parts.

#### Note

- **Wash** breastpump kit parts after each use.
- **Sanitize** breastpump kit parts once daily.
- **Take care** not to damage parts of the pump kit during cleaning.

### 2.3 Sanitize Daily

Wash hands thoroughly.

#### Options for sanitizing:

Quick Clean Micro-Steam Bag:

You may sanitize your kit parts by following the instructions on Medela's Quick Clean™ Micro-Steam™ bag.

Do **NOT** Micro-Steam the Lactina® piston and cylinder. 

Boiling:

Separate all parts that come in contact with your breast and breastmilk.

- Fill a pot with enough water to cover all parts.
- Bring water to a boil.
- Place parts in boiling water for 10 minutes.
- Allow water to cool and gently remove parts from water with tongs.
- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts when not in use. Do **NOT** store wet or damp parts.

#### Note

- Visit [www.medela.com](http://www.medela.com) for additional cleaning products.

## 2. Cleaning (cont.)

### 2.4 Piston/Cylinder Cleaning

Care for your piston and cylinder after every few pumping sessions with the Lactina breastpump or manual breastpump.

- Wash the cylinder, seal, and vacuum regulator ring with warm soapy water.
- Rotate the piston in the cylinder.
- If your piston/cylinder comes in contact with breastmilk remove and wash with breastpump kit.
- Do not use Quick Clean™ Micro-Steam™ bag to sanitize the piston and cylinder.

### 2.5 Tubing Cleaning

Inspect tubing after each pumping session. Clean tubing if there is any sign of condensation and/or milk.

- Rinse and run cold water through tubing to remove breastmilk.
- Wash tubing in warm soapy water.
- Rinse tubing with clear water.
- Hang to air dry.
- You may also dry tubing by attaching it to pump and running for 2 minutes or until dry. This will eliminate condensation that naturally results from humidity.
- In addition to washing, you may sanitize by following the instructions in section 2.3.
- Tubing may appear cloudy after boiling or microsteaming.

### 3. Storing breastmilk

#### Caution

- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water.
- Do not microwave breastmilk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving. (Microwaving can also change the composition of breastmilk.)

#### Note

- As soon as you are finished pumping, place the breastmilk bottle or breastmilk storage bag in the cooler bag with the ice pack.
- Transfer breastmilk bottles or breastmilk storage bag to the refrigerator or freezer once you are home.

#### Freshly Expressed Breastmilk Storage Guidelines (For Healthy Term Babies)

Room Temperature	Cooler with Ice Pack	Refrigerator	Freezer	Thawed Breastmilk
4–6 hours at 66–78 °F (19–26 °C)	24 hours at 59 °F (15 °C)	3–8 days at 39 °F or lower (4 °C)	6–12 months 0–4 °F (-18–-20 °C)	use within 24 hrs

References: [www.BreastmilkGuidelines.com](http://www.BreastmilkGuidelines.com)

## 4. Troubleshooting

<p><b>Low or No Suction</b></p>	<p><b>Valve and Membrane Check</b></p> <ul style="list-style-type: none"> <li>Remove and separate valve from membrane</li> <li>Inspect the valve for chips or cracks. Inspect the membrane for holes or tears. If flawed or damaged replace valve and/or membrane prior to pumping</li> <li>Wash valve and membrane per Cleaning instructions in section 2.</li> <li>Reassemble. Make sure the membrane lays flat on the valve, and if it does not, replace the valve and/or membrane prior to pumping</li> </ul> <p><b>Breastshield and Breastshield Connector Check</b></p> <ul style="list-style-type: none"> <li>Inspect breastshield and breastshield connector to make sure there are no cracks and they securely attach to each other</li> </ul> <p><b>Other Checks</b> Inspect the following to check for damage and secure connections.</p> <ul style="list-style-type: none"> <li>Tubing adapter to breastshield connector</li> <li>Inspect Symphony membranes. If membranes appear to be worn or have a tear they need to be replaced (Article # 8200094).</li> <li>Inspect piston/cylinder on Lactina. Refer to Piston/Cylinder Cleaning in section 2.4 for further instructions.</li> <li>Test vacuum. If suction is not improved, contact Medela Customer Service at 1-800-435-8316.</li> </ul>
<p><b>Condensation or milk in tubing</b></p>	<ul style="list-style-type: none"> <li>Refer to “Tubing cleaning” section 2.5.</li> </ul>
<p><b>No “let-down” or breastmilk expression</b></p>	<ul style="list-style-type: none"> <li>Ensure your breastpump kit is assembled correctly and that there is suction.</li> <li>Relax and take a 10-15 minute break if let down is not achieved after 2 consecutive pumping sessions.</li> <li>Consult with your healthcare provider or breastfeeding specialist if expression does not occur.</li> </ul>

If assistance is still needed after reviewing this booklet, please call Medela Customer Service at (800) 435-8316 or (815) 363-1166.

## 5. Supplemental Information

Following are some common breastfeeding related conditions. If you experience any of these symptoms, contact a healthcare professional or breastfeeding specialist.

	Symptom	Potential Causes
<p><b>Engorgement</b></p>	<p>Breasts are hard and uncomfortable, possibly with reddened areas.</p>	<p>Milk not draining properly from breast. Milk can drain back into the tissue causing areas to swell and become tender.</p>
<p><b>Blocked/ Plugged Ducts</b></p>	<p>Area of breast will look reddened and may be tender to touch.</p>	<p>Milk not being drained from a specific duct. The area becomes ‘clogged’ up and milk is then prevented from flowing.</p>
<p><b>Mastitis</b></p>	<p>It often follows on from engorgement but can just occur suddenly. The initial symptoms are similar to flu – fatigue, headache and muscle aches, fever and localized breast tenderness. It can be in one or even both breasts and needs immediate treatment.</p>	<p>A bacterial infection in the breast tissue which is often accompanied by cracked nipples.</p>

A Healthcare Professional or Breastfeeding Specialist can give you guidance in breastfeeding your baby and using a breastpump. Their guidance is valuable in terms of the lifetime of health benefits for both you and your baby. A listing of Lactation Professionals are available at [www.medelalocator.com](http://www.medelalocator.com) or call 1-800-TELL-YOU (1-800-835-5968).

Medela, Inc.  
1101 Corporate Drive  
McHenry, IL 60050, USA  
Tel.: (800) 435-8316 or (815) 363-1166  
Fax: (815) 363-1246  
Email: [customer.service@medela.com](mailto:customer.service@medela.com)

[www.medela.com](http://www.medela.com)

[www.medela.com](http://www.medela.com)

Medela, Inc.  
1101 Corporate Drive  
McHenry, IL 60050, USA  
Tel.: (800) 435-8316 o (815) 363-1166  
Fax: (815) 363-1246  
Correo electrónico: [customer.service@medela.com](mailto:customer.service@medela.com)

Medela, Symphony and Lactina are registered trademarks of Medela Holding AG.  
PersonalFit, Quick Clean and Micro-steam are trademarks of Medela, Inc.  
© 2013 Medela, Inc. Printed in the U.S.A.

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